

PURE DRAFT  
*Serene*<sup>™</sup>  
BREAST IMPLANT




**Structured Saline Breast Implant**

**REDEFINING POSSIBILITIES**

# Uniquely designed saline implant with an **internal structure**

The **Serene Structured Saline Breast Implant** was engineered to provide a more customized fit based on your patient's desired needs and outcomes.

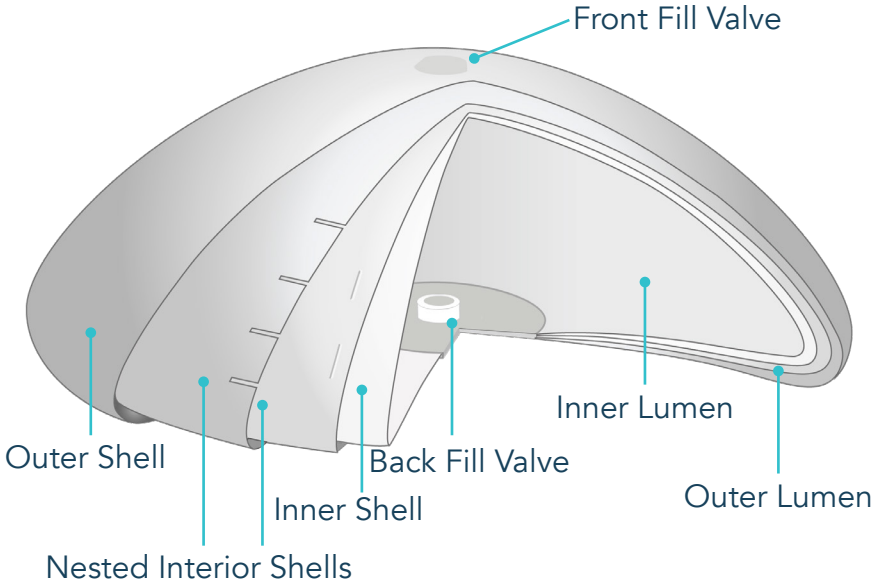

## Thoughtfully engineered, robotically manufactured.

**Low Capsular Contracture and Low Deflation Rates<sup>1</sup>.**

1: Reference: Nichter LS, Hardesty RA, Zimmerman TJ. Ideal Implant Structured Breast Implants: Core Study Results Through 10 Years. *Plast Reconstr Surg.* 2023; 152(3):424e-432e. doi 10.1097/PRS00000000000010312

# What makes the **Serene Structured Breast Implant** different?

- **Dual Fill Chambers Ensure:**
  - A stable base
  - Adjustable height and projection
  - Upper pole fullness
- **Interior Shells:**
  - Control the movement of saline
  - Help minimize edge collapse
  - Minimize rippling and wrinkling
- **Made in USA:**
  - Generation 2 technology
  - Robotic dipping creates a more uniform shell
- **No MRI or Scans** required to detect implant rupture
- **Pure Confidence Warranty:**
  - Lifetime coverage for implant replacement due to deflation/rupture

**PURE DRAFT**  
*Serene*<sup>™</sup>  
**BREAST IMPLANT**

# Providing patients with a state-of-the-art structured saline implant option



Moderate Profile at Minimum Fill  
Full Profile at Maximum Fill

EMPTY + INNER + OUTER = TOTAL VOLUME

IMPLANT SIZE	EMPTY IMPLANT VOLUME	INNER LUMEN SALINE	OUTER LUMEN SALINE	TOTAL IMPLANT VOLUME	BASE DIAMETER	PROJECTION
<b>210cc</b>	30cc	120cc	60 - 85cc	210 - 235cc	10.1 - 10.0cm	3.5 - 4.3cm
<b>240cc</b>	33	142	65 - 95	240 - 270	10.5 - 10.4	3.6 - 4.5
<b>270cc</b>	35	165	70 - 105	270 - 305	11.0 - 10.8	3.8 - 4.7
<b>300cc</b>	37	188	75 - 115	300 - 340	11.4 - 11.2	3.9 - 4.9
<b>335cc</b>	52	188	95 - 135	335 - 375	11.9 - 11.7	4.0 - 5.1
<b>370cc</b>	56	214	100 - 145	370 - 415	12.2 - 12.0	4.1 - 5.2
<b>405cc</b>	60	235	110 - 160	405 - 455	12.5 - 12.4	4.2 - 5.4
<b>440cc</b>	64	261	115 - 170	440 - 495	12.9 - 12.8	4.3 - 5.6
<b>475cc</b>	68	287	120 - 180	475 - 535	13.3 - 13.1	4.4 - 5.7
<b>515cc</b>	72	318	125 - 190	515 - 580	13.6 - 13.4	4.5 - 5.8
<b>555cc</b>	76	344	135 - 205	555 - 625	13.9 - 13.8	4.6 - 6.0
<b>595cc</b>	94	346	155 - 230	595 - 670	14.3 - 14.2	4.7 - 6.1
<b>635cc</b>	102	373	160 - 235	635 - 710	14.6 - 14.5	4.8 - 6.2
<b>675cc</b>	110	405	160 - 240	675 - 755	14.9 - 14.8	4.9 - 6.3

**Learn More | Get Serene**

Email [orders@biminihealthtech.com](mailto:orders@biminihealthtech.com)  
or call **858-348-8050**.

PURE DRAFT  
*Serene*<sup>™</sup>  
BREAST IMPLANT



[serenebreastimplants.com](http://serenebreastimplants.com)

The sale and distribution of this device is restricted to users and/or user facilities that provide information to patients about the risks and benefits of this device in the form and manner specified in the approved labeling provided by Bimini Health Tech.

**WARNING:**

- Breast implants are not considered lifetime devices. The longer people have them, the greater the chances are that they will develop complications, some of which will require more surgery.
- Breast implants have been associated with the development of a cancer of the immune system called breast implant-associated anaplastic large cell lymphoma (BIA-ALCL). This cancer occurs more commonly in patients with textured breast implants than smooth implants, although rates are not well defined. Some patients have died from BIA-ALCL.
- Patients receiving breast implants have reported a variety of systemic symptoms such as joint pain, muscle aches, confusion, chronic fatigue, autoimmune diseases and others. Individual patient risk for developing these symptoms has not been well established. Some patients report complete resolution of symptoms when the implants are removed without replacement.

**Bimini Health Tech**

+1-858-348-8050  
[info@biminihealthtech.com](mailto:info@biminihealthtech.com)  
[www.biminihealthtech.com](http://www.biminihealthtech.com)